



BREEZESUITE™ 8.4 Service Pack 2 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiorespiratory diagnostic software version 8.4 and is intended to be installed on any computer that has BreezeSuite 8.4 already installed. BreezeSuite 8.4 Service Pack 2 can be installed on workstations, but not on BreezeSuite Servers.

Minimum System Requirements

- BreezeSuite cardiorespiratory diagnostic software version 8.4

Warning: Installing Service Pack 2 over BreezeSuite software version 8.4 will over-write the translation files.

NOTE: For multiuser installations, Service Pack 2 is only applied to workstations. Service Pack 2 is not applied to the BreezeSuite Multiuser™ Server installation.

INSTALLATION

1. Close all applications.
2. Place the USB Flash Drive into the computer. Using Windows Explorer, browse to the USB Flash Drive. If downloaded from the MGC Diagnostics website, extract/unzip the BreezeSuite84SP2.zip file.
3. Double click on the BreezeSuite84SP2.exe file on the USB Flash Drive, or from the extracted files if downloaded from the MGC Diagnostics website.
4. After the software installation is run, it will begin to install the appropriate files to your hard drive.
5. After the files have been installed, remove the USB Flash Drive from the computer if applicable.

LANGUAGE TRANSLATIONS

Service Pack 2 has been translated into: French, Spanish, Portuguese, and Russian.

1. Follow the Installation Instructions above before proceeding.
2. With the Service Pack 2 USB Flash Drive in the computer, browse to the **Translations** folder on the USB Flash Drive.
3. If the files were downloaded from the MGC Diagnostics website, browse to the **Translations** folder of the file that was extracted.
4. Double click on the appropriate language executable file.
5. At the “Location To Save Files” screen select **Next**.
6. At the “Overwrite Protection” screen select **Yes To All**.
7. After the files have been extracted and copied select **Finish**.

Changes for Service Pack 1

- Updated text reports to include header and history section.
- Corrected issue with clinical link between BreezeSuite and Welch Allyn CardioPerfect ECG software.
- Corrected issue with blank pages on a GX report.
- Corrected issue with PF/GX trend data not updating in Breeze Review in between reviewing patient visits.
- Corrected issue with adjusting dead space values on GX scripts.
- Improved interface between FOT and BreezeSuite.

Changes for Service Pack 2

- Integration of X-Scribe 4.0 CPI (cardiopulmonary interface).
- Corrected issues with active directory integration.
- Corrected issues with FOT graphs and labeling.
- Updated default FOT report and screen configuration graphs.
- Corrected issue related to auto filters and staff login settings.
- Corrected issue with Breeze Order Entry that allows users to cancel orders regardless of current order status.