



## RELEASE NOTICE

This software is for the CPX Express and is intended to be installed on any CPX Express System.

### Minimum System Requirements

- CPX Express v1.2

### Changes Made in CPX EXPRESS v1.2 SP1 Software Release

- Added the ability to export test data to BreezeSuite .cow file format for importing into a BreezeSuite database.
- Updated the “Splash Screen” to MGC Diagnostics branding.
- Updated the calculation of Vd/Vt (see Product Bulletin 140434-001 for further details).

## INSTALLATION INSTRUCTIONS

1. Download the CPX Express v1.2 Service Pack 1 (SP1) software. The software can be downloaded from:  
<http://www.mgcdiagnostics.com/support>
2. Copy the software to a USB Flash Drive.
3. Connect the USB Flash Drive to a USB port on the CPX Express System.
4. Run the CPX Express v.1.2 SP1 software from the USB Flash drive.
5. After the software is installed, remove the USB Flash Drive from the CPX Express system.