



BREEZESUITE™ 8.4 Service Pack 5 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiorespiratory diagnostic software version 8.4 and is intended to be installed on any computer that has BreezeSuite 8.4 already installed. BreezeSuite 8.4 Service Pack 5 can be installed on workstations, but not on BreezeSuite Servers.

Minimum System Requirements

- BreezeSuite cardiorespiratory diagnostic software version 8.4

Warning: Installing Service Pack 5 over BreezeSuite software version 8.4 will over-write the translation files.

NOTE: For multiuser installations, Service Pack 5 is only applied to workstations. Service Pack 5 is not applied to the BreezeSuite Multiuser™ Server installation.

INSTALLATION

1. Close all applications.
2. When downloaded from the MGC Diagnostics website, extract/unzip the BreezeSuite84SP5.zip file.
3. Double click on the BreezeSuite84SP5.exe file from the extracted files downloaded from the MGC Diagnostics website.
4. After the software installation is run, it will begin to install the appropriate files to your hard drive.

LANGUAGE TRANSLATIONS

The translation files for Italian, Spanish, French, Russian and Portuguese are included in this packet for Service Pack 5.

Changes for Service Pack 1

- Updated text reports to include header and history section.
- Corrected issue with clinical link between BreezeSuite and Welch Allyn CardioPerfect ECG software.
- Corrected issue with blank pages on a GX report.
- Corrected issue with PF/GX trend data not updating in Breeze Review in between reviewing patient visits.
- Corrected issue with adjusting dead space values on GX scripts.
- Improved interface between FOT and BreezeSuite.

Changes for Service Pack 2

- Integration of X-Scribe 4.0 CPI (cardiopulmonary interface).
- Corrected issues with active directory integration.
- Corrected issues with FOT graphs and labeling.
- Updated default FOT report and screen configuration graphs.
- Corrected issue related to auto filters and staff login settings.
- **Corrected issue with Breeze Order Entry that allows users to cancel orders regardless of current order status.**

Changes for Service Pack 3

- Corrected issue with ExFVL default graph configuration.
- Corrected assert error issue indicating decrypted passwords do not exist in database if password entries are different between local and shared database.
- Corrected issue with External Device Calibration record when refreshing local database.
- Corrected issues related to DBTools identifying active directory server based on domain.

Changes for Service Pack 4

- Corrected an issue where the connection string used for Standalone mode (ie Citrix/no local database) does not attempt to reconnect when the connection is lost to the database. BreezeSuite will now close and attempt to reload the connection.

Changes for Service Pack 5

- Increased the maximum password expiration from 90 to 180 days.
- Corrected issue with BRP Graph label for first stage marker.
- The “Consult” button in Breeze Review has been relabeled to “Interpretation”.
- Corrected issue with identifying VO2 max.
- When displaying a list of patients, the first column which displays the number of patients/visits will now automatically adjust its’ width, based on the size of the number.
- Corrected an issue with decimal places for height not updating on reports.
- Corrected an issue when combining patient files with duplicate visits or when combining staff files.
- For Text Report, added the ability to specify the width of data columns, specify the width of consult and comments, and remove carriage returns.
- Added the ability to set the limit of the number of patient visits displayed.
- Increased the Export path character length.
- Added a global “Options/Patient/ForcePatientListRefresh” that determines if the patient/visit list should be refreshed every time the patient/visit list is displayed.