



BREEZESUITE™ 8.3 Service Pack 3 Release & Installation Notice

This software is a Service Pack for BreezeSuite™ cardiorespiratory diagnostic software version 8.3 and is intended to be installed on any computer that has BreezeSuite 8.3 already installed.

Minimum System Requirements

- BreezeSuite cardiorespiratory diagnostic software version 8.3

Warnings:

- Installing Service Pack 3 over BreezeSuite software version 8.3 will over-write the translation files.
- If a software repair is performed and BreezeSuite 8.3 is reinstalled, Service Pack 3 must be immediately reinstalled.

NOTE:

For multiuser installations, Service Pack 3 is applied only to workstations. Service Pack 3 is not applied to the BreezeSuite Multiuser™ Server installation.

INSTALLATION

1. Close all applications.
2. When downloaded from the MGC Diagnostics website, extract/unzip the BreezeSuite84SP5.zip file.
3. Right click on the BreezeSuite83SP3.exe file from the extracted files downloaded from the MGC Diagnostics website and select “Run as Administrator”.
4. Follow the on-screen prompts to install the appropriate files to your hard drive.

LANGUAGE TRANSLATIONS

Service Pack 3 (147926-502) is released in English only.
The translation files for this Service Pack will be released as separate files.

Changes for Service Pack 1

- Added Analog I/O module support for newly repackaged Ultima Series cardiorespiratory diagnostic systems
- Corrected issue with incorrect PF data showing on GX patients in Breeze Review
- Improved performance in calculating GX and PF predicted values
- Corrected issue with New Visit options for Orders
- Corrected issue with PDF report selection in Breeze Review

- Correct issue with scissor valve operations
- Corrected communication issue related to Lode ergometers utilizing legacy or obsolete versions of Lode ergometer firmware/communications
- Correct issue with DLco calibration failing when a MIP/MEP calibration is performed prior to a DLco calibration

Changes for Service Pack 2

- Improved communication protocol with Trackmaster 428 treadmill
- Corrected an issue in Breeze Review which caused the software to shut down when reports containing test view pages were exported after a visit was marked as Completed+Posted+Locked
- Corrected an issue where recently converted Carefusion (SensorMedics) patient files are Auto Exporting multiple visits

Changes for Service Pack 3

- Corrected an issue in which the computer would not communicate with MGC testing systems after Windows 10 Feature Upgrade, version 1803 was installed.