



This software is an update to Resmon Pro FOT V3 version 21.1.1 or older and can be installed directly on top of these older versions.

NOTE: Once a system has been updated to FOT version 21.3.0, the software cannot be rolled back to a previous version.

Installation Procedure

1. Download the Resmon Pro FOT V3 version 21.3.0. file from the MGC Support website.
2. Copy this file (**resmonpro-21.3.0-update.tar.bz2**) to a flash drive.
3. Insert the flash drive into any of the USB ports on the back of the Resmon PRO.
4. Turn on the device.
5. From the main menu select “ADMIN” then “SOFTWARE UPDATE” and let the system find the USB key and the available software updates.
6. Select **resmonpro-21.3.0-update** and follow the instructions until the device automatically turns itself off.