



This software is an update to FeNObreath devices running v3.1 firmware and can be installed directly on top of the older version.

NOTE: Once a system has been updated to version 3.53, the software cannot be rolled back to a previous version.

Firmware Update Procedure

- 1.1 Press and hold the on/off button once to switch the monitor on and wait up to 60 seconds for the FeNObreath device to stabilize.
- 1.2 Connect the FeNObreath to the PC via the USB (the computer must have access to the internet).
- 1.3 Download the Bedfont Firmware update tool from the connected PC following the onscreen instructions (Allow tool to update if required)
 - 1.3.1 The update tool can be downloaded from the following URL:
https://www.bedfont.com/downloads/Firmware_Updater.exe
- 1.4 Open the downloaded Update tool.
- 1.5 After the tool has opened and the unit information has loaded click the 'Browse' button and select the firmware file – FeNObreathFirmwareV3.53.ftb'
- 1.6 Select 'Open'
- 1.7 Start the firmware update after the button is clickable (Once the firmware update has started, do not disconnect the USB.)
 - 1.7.1 The unit will power off and the firmware will start updating. The tool will display the installation progress in the bottom right.
 - 1.7.2 After the installation reaches 100% the device will power on and display the updating images. Once the images have updated, the device will power off again to continue the firmware installation.
- 1.8 Wait for the tool to finish before disconnecting the USB.
- 1.9 Restart the Device