

FeNObreath®

NO analyzer




MEASURE EXHALED BREATH NITRIC OXIDE FOR AIRWAY INFLAMMATION WITH THE FeNObreath® PORTABLE MONITOR

Based on proven, validated Bedfont® Scientific Ltd. FeNO technology, MGC Diagnostics International introduces the newest addition to the Exhaled Nitric Oxide lines.

FeNObreath® portable device, for use on pediatric and adult patients:

- Portable, light, transportable, battery operated device can be used in the hospital or at remote locations (schools, work places, screening programs etc.), stores patients results in its memory for later download.
- Single patient mouthpiece for safe testing at economical cost/patient: the FeNObreath® mouthpiece is a single patient use and is specifically designed with an integrated filter to remove > 99% of airborne bacteria and > 96% of viruses.
- Downloads, integrates data in the Expair II PC software for storage, reporting, trending, integrate data with other MGCD devices. Expair can be used to display in Real time the FeNObreath® measurement.
- FeNO results can be added to Spirometry and other pulmonary function test results and reports, as well as to the table-top FeNO+ data output, completing the powerful, exclusive "asthma diagnostics station" that features as an exclusive system.
- In combination with Resmon Pro Full V3 FOT (Forced Oscillatory Technique) results allows unique combined obstruction and inflammation detection, for asthma early detection, management programs and screening.
- Color touch-screen display, easy intuitive test with simple menu driven
- procedure:

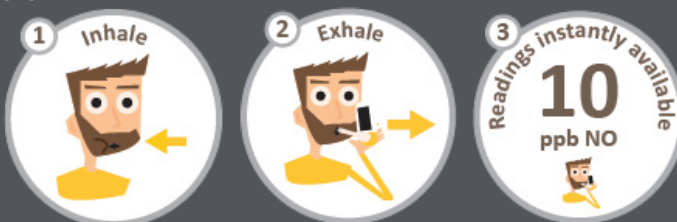

BENEFITS OF PERFORMING FeNO TESTS:



- Non-invasive, quick and easy to perform¹
- Shows patient's response to treatment, enabling the correct prescription of medication and safer/monitored adjustments
- Shows patient adherence to treatment²
- Assists in identifying patients who may or may not require on-going treatment³
- Aids in differentiating between allergic (eosinophilic) and non-allergic asthma⁴

1. Andrew D. Smith, Jan O. Cowan, Sue Filsell, Chris MacLachlan, Gabrielle Monti-Sheehan, Pamela Jackson and D. Robin Taylor. *Diagnosing Asthma: Comparisons between Exhaled Nitric Oxide Measurements and Conventional Tests.* Am J Respir Crit Care Med Vol 169. pp 473-478, 2004.
2. Beck-Ripp J, Griesem, Koring C, Pasqualoni B, Butler P. *Changes of exhaled nitric oxide during steroid treatment of childhood asthma.* Eur Respir J 2002;19:1015-1019.
3. D R Taylor, MW Pinenburg, A D Smith and J C D Jongste. *Exhaled nitric oxide measurements: clinical application and interpretation.* Thorax 2006;61:817-827.
4. Coumou HBel E. *Improving the diagnosis of eosinophilic asthma [Internet].* Taylor and Francis online. 2017 [cited 15 March 2017]. Available from: <http://www.tandfonline.com/doi/10.1080/17476348.2017.1236688>

Technical Specifications:

Concentration range		5 - 500ppb
Display		Full color Touchscreen
Detection principle		Electrochemical sensor
Repeteability		± 5ppb of measured value ≤ 50ppb ± 10% of measured value > 50ppb
Accuracy		± 5ppb of measured value ≤ 50ppb ± 10% of measured value > 50ppb
Power	FeNObreath® monitor	1 x main rechargeable Li-ion battery Approx. 100 uses on fully charged battery 2 x Li-ion coin cell battery - Approx 5 years Input: 5V 0.5A
	FeNObreath® Dock	Mains powered Input: 5V 0.5A Output: 5V 0.5A
	Plug	Input: 100 - 240V - 50/60Hz, 0.2A Output: 5V 1.0A
T50 response time		≤ 10 seconds
Operating temperature		15 - 30°C
Storage/transport temperature		0 - 50°C
Operating/storage/transport pressure		920 - 1080mbr
Operating humidity		20 - 80% non-condensing
Storage/transport humidity		9 - 95% non-condensing
Sensor operating life		5 years (Subject to correct servicing)
Sensor sensitivity		1ppb
Sensor drift		< 5% per annum
Dimensions		Approx. 90 x 159 x 59mm Approx. 3,54 x 6,25 x 2,32 inches
Weight		Approx. 400g
Materials	FeNObreath® monitor	Case polycarbonate/abs blend
	FeNObreath® Dock	SteriTouch® anti-microbial additive
	FeNObreath® mouthpiece	Polypropylene
Breath test time		Adult: 12 seconds Child: 10 seconds Ambient: 30 seconds
Warm-up time		≤ 60 seconds
Maximum ambient operating level		350ppb NO
CO cross interference		45ppm ≤ 17.6ppb
Note		Exhaled flow during FeNO measurement at 50ml/sec ± 10% at 10cm H2O

DESIGNED, DEVELOPED AND MANUFACTURED BY:



Bedfont Scientific Ltd, UK

Part# 0123FEBREVEN

MGC DIAGNOSTICS CORPORATION, through its subsidiary Medisoft SA
350 Oak Grove Parkway, St. Paul, Minnesota USA 55127-8599
Medisoft S.A. P.A.E de Sorinnes, Rue du Clairon 5
5503 Sorinnes BELGIUM

© 2023 MGC Diagnostics Corporation or one of its affiliates. All rights reserved.

All specifications subject to change without notice. Products may vary from those illustrated.

MGC Diagnostics and its affiliates are equal opportunity/affirmative action employers committed to cultural diversity in the workforce.

