



PREPARACIÓN DEL PACIENTE PARA LA PRUEBA DE FeNO



SE TRATA DE UNA SUGERENCIA DE PROTOCOLO; PUEDE HABER OTROS PROTOCOLOS DISPONIBLES.

24 HORAS ANTES DE LA PRUEBA:

Se recomienda evitar el consumo de alcohol durante las 24 horas previas a la prueba ¹.

12 HORAS ANTES DE LA PRUEBA:

Se recomienda evitar fumar 12 horas antes de la prueba ¹.

Se recomienda evitar el consumo de alimentos ricos en nitratos, entre los que se incluyen los siguientes ²:

Rúcula	Nabo	Zanahoria
Espinaca	Repollo	Patata
Lechuga	Judía verde	Ajo
Rábano	Puerro	Pimiento dulce
Remolacha	Cebolleta	Pimiento verde
Col china	Pepino	

DÍA DE LA PRUEBA:

El día de la prueba, se recomienda evitar lo siguiente 1 hora antes de la prueba ²:

Comer	Fumar
Beber	Realizar ejercicio

*Las Directrices ATS/ERS incluidas en las referencias a continuación, establecen que las actividades enumeradas pueden afectar a las lecturas FeNO y, por lo tanto, MGC Diagnostics ha recomendado abstenerse de estas actividades solo durante un determinado período de tiempo.

BIBLIOGRAFÍA:

1. ATS/ERS Recommendations for Standardized Procedures for the online and offline measurement of exhaled lower respiratory nitric oxide and nasal nitric oxide, 2205. American Journal of Respiratory and Critical Care Medicine. 2005; 171 8): 912-930.
2. Lidder S, Webb A. Vascular effects of dietary nitrate (as found in green leafy vegetables and beetroot via the nitrate--nitrite-nitric oxide pathway. British Journal of Clinical Pharmacology. 2013; 75(3): 677-696.



MGC DIAGNOSTICS CORPORATION, through its subsidiary Medisoft S.A.
350 Oak Grove Parkway St. Paul, Minnesota USA 55127-8599

 Medisoft S.A. P.A.E de Sorinnes, Rue du Clairon 5
5503 Sorinnes BELGIUM

© 2024 MGC Diagnostics Corporation or one of its affiliates. All rights reserved.

All specifications subject to change without notice. Products may vary from those illustrated.

MGC Diagnostics and its affiliates are equal opportunity/affirmative action employers committed to cultural diversity in the workforce.

Part# 060178-003 RevA

