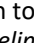
Quick Start Guide

Taking a Breath Test

1. Press and hold the on/off button once to switch the monitor on and wait up to 60 seconds for the monitor to stabilise.
2. Attach a new mouthpiece to the unit and select either the adult  or child  breath test.
NOTE: To cancel the breath test, press the  icon.
3. As prompted onscreen by  or , inhale until prompted to exhale.
NOTE: Do not inhale through the mouthpiece.
4. When the exhale icons appear,  or , keep the monitor upright and blow gently into the mouthpiece. Make sure you are not covering the vent holes.
NOTE: The exhalation time is approximately 12 seconds for an adult and 10 for a child.
5. The onscreen flow meter will guide you on the exhalation rate: continue blowing until the green tick  appears onscreen to signify a successful test.
NOTE: If you blow outside of the exhalation guidelines, the test will beep, before signifying a fail, and a red cross  will appear. Touch  to retry the test or  to view the result.
6. Following the tick, a result will appear on the screen, in ppb.
7. You may save this result by pressing . Alternatively, you can return to the home screen by pressing .
8. If you choose to save your results you will be shown a list of patient profiles. Select a previously set profile to add your result to, or select an empty profile to set a new one. Set the profile by typing in a name or reference, and press .
9. Remove and dispose of the mouthpiece in accordance with local waste and disposal guidance.
10. If no further tests are required, press and hold the on/off button once to turn the FeNObreath™ off and store on FeNObreath™ Dock to charge unit.

Note: A PIN is required to save the reading to a patient profile, the default PIN is 0000
It is highly recommended that you change the PIN to a 4 digit number that you will remember. If you have not entered your PIN in the last 30 minutes, you will be asked to enter it before you can access the patient profiles.
If your PIN is entered incorrectly, you will be notified and the screen will refresh for you to re-enter your PIN.
If you have forgotten your PIN, please contact MGC Diagnostics or your local distributor to reset it.

For a full copy of the manual please contact your local distributor

Bedfont® Scientific Ltd.
Station Road, Harrietsham, Maidstone,
Kent, ME17 1JA England
Tel: +44 (0)1622 851122 Fax: +44 (0)1622
854860
Email: ask@bedfont.com
Web: www.bedfont.com

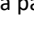
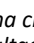
MGC Diagnostics Corporation
and its subsidiary Medisoft SA
350 Oak Grove Parkway, St. Paul, MN USA, 55127
Tel: +1 800 950 5597
1 Route de la Voie Cuivrée 5503 Sorinnes, Belgium
Tel: +32(0) 82 22 30 20
Web: www.mgcdiagnostics.com

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